


# ICON 20hCG AND ICON 25hCG

ONE STEP TESTS FOR THE EARLY DETECTION OF PREGNANCY



## INTERPRETING ICON 20hCG AND ICON 25 hCG TESTS >

### ICON 20hCG and 25hCG **Positive** Test

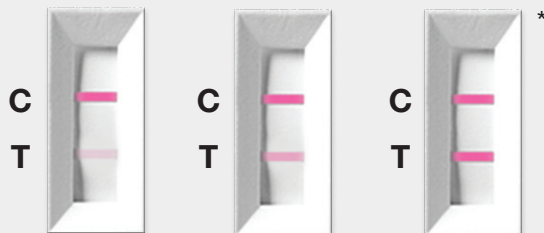

**Positive Test Result:** Two red or pink color lines, one each at the Test Position (T) and the Control Position (C). Line color intensity can be light to strong as illustrated above.

*Note: Any hint of a red or pink line in the test line area is a positive test result.*

### ICON 20hCG and 25hCG **Negative** Test



**Negative Test Result:** Only one red or pink color line at the Control Position (C). No apparent red or pink line in the Test Position (T).

### READ TIME:

- > **ICON 20hCG** Read test results at 3 to 5 minutes. Confirm the negative results at 5 minutes.
- > **ICON 25hCG** Read the result at 3 minutes when testing a urine sample, or at 5 minutes when testing a serum sample.

To order product, contact your medical supply distributor.

For technical assistance regarding rapid test kits, contact Technical Marketing at 800-877-6242 or email [askpcd@beckman.com](mailto:askpcd@beckman.com). [www.beckmancoulter.com/rapids](http://www.beckmancoulter.com/rapids)

\* Above illustrations are an artist's renditions. Each illustration shows one test and one control line from a single sample as displayed on a single ICON hCG test device.

Beckman Coulter, the stylized logo and ICON are trademarks of Beckman Coulter, Inc. and are registered with the USPTO.

For Beckman Coulter's worldwide office locations and phone numbers, please visit [www.beckmancoulter.com/contact](http://www.beckmancoulter.com/contact)



FL-18498A B2013-14478  
© 2013 Beckman Coulter, Inc.

[www.beckmancoulter.com](http://www.beckmancoulter.com)

PRINTED IN U.S.A.